STD/HPV Transport Medium

for Sexually Transmitted Disease(STD)/Human Papilloma Virus(HPV)

Cervix-Swab™
Cervical Cyto-Brush, Endo/Exo Cervical Flocked Swab & Curette

URETHRAL SWAB SPECIMENS COLLECTION

* Do not urinate for at least one hour prior to collecting specimen.

- (1) Peel open the Urethral swab pouch and hold the swab, the upper part of breakpoint. Please use a new swab if you touch the soft tip or lower part of breakpoint.
- (2) Insert Urethral Swab 2~4 cm into the urethra and gently rotate the swab clockwise for 2 to 3 seconds in the urethra for adequate specimen sampling and remove the swab carefully.
- (3) After remove the cap from the medium tube, immediately place the specimen collection swab into the transport medium.
- (4) Carefully break the swab shaft against the top side of the tube at the breakpoint and be careful to avoid splashing of contents.
- (5) Recap the specimen transport medium tube tightly and homogenize the sample medium by shaking from side to side.
- (6) Record all the information about the specimen on the label. And send it to the laboratory according to their specifications as soon as possible.

THROAT/PHARYNGEAL SWAB SPECIMENS COLLECTION

- (1) Peel open the Throat/Pharyngeal swab pouch and hold the swab, the upper part of breakpoint. Please use a new swab if you touch the soft tip or lower part of breakpoint.
- (2) By using the specimen collection swab, Please sample the bilateral posterior pharynx, both tonsils and the uvula.
- (3) After sampling, remove the cap from the transport medium tube and immediately place the specimen swab into the transport medium. And carefully break the swab shaft against the top side of the tube at the breakpoint and be careful to avoid splashing of contents.
- (4) Recap the specimen transport medium tube tightly and homogenize the sample medium by shaking from side to side.
- (5) Record all the information about the specimen on the label. And send it to the laboratory according to their specifications as soon as possible.

STD / HPV Swab Selection by symptom

	Women	Men
HPV (Human Papilloma Virus)	Cervical Swab (Endo/Exo Cervical Swab / Cervical Brush)	-
Cervicitis (자궁경부염)	Vaginal Swab / Cervical Swab / Urine Swab	-
Urethritis (요도염)	Urine Swab	Urethral Swab / Urine Swab
Proctitis (직장염)	Rectal Swab	Rectal Swab
Vaginal discharge	Vaginal Swab / Cervical Swab / Urine Swab	-

Cervix-Swab™
Cervical Cyto-Brush, Endo/Exo Cervical Flocked Swab & Curette

STD/HPV Transport Medium

for Sexually Transmitted Disease(STD)/Human Papilloma Virus(HPV)

HPV/STD SAMPLE COLLECTION,
TRANSPORT & PRESERVATION SYSTEM

STD/HPV Transport Medium How to Use Guide :

- **Product Name** : STD/HPV Transport Medium
- **Cat.No** : HPV-B-CB1-N1P, HPV-B-CB1C-N1P, HPV-B-NFS2-N1P
- **Certificate.No** : 15-4092

products : HPV-B-CB1-N1P, HPV-B-CB1C-N1P, HPV-B-CB2-N1P, HPV-B-CB2C-N1P, HPV-B-NFS2-N1P, HPV-B-NFS7-N1P, HPV-B-NFS8-N1P, HPV-C-CB1-N1P, HPV-C-CB1C-N1P, HPV-C-CB2-N1P, HPV-C-CB2C-N1P, HPV-C-NFS2-N1P, HPV-C-NFS7-N1P, HPV-C-NFS8-N1P, NPTB-CB1-N1P, NPTB-CB1C-N1P, NPTB-CB2-N1P, NPTB-CB2C-N1P, NPTB-NFS2-N1P, NPTB-NFS7-N1P, NPTB-NFS8-N1P, NPCT-CB1-N1P, NPCT-CB1C-N1P, NPCT-CB2-N1P, NPCT-CB2C-N1P, NPCT-NFS2-N1P, NPCT-NFS7-N1P, NPCT-NFS8-N1P

1. INTENDED USE

Sexually Transmitted Disease(STD) / Human Papilloma Virus(HPV) Collection Swab containing specimen collection swab and transport medium can be used for culture, rapid antigen testing and molecular based assay with several types of transport media.

Also, STD/HPV DNA preservation & transport kit is used for DNA preservation and transportation to store in safety at room temperature after specimen collection with Vaginal, Cervical, Urethral, Rectal swab or Throat swab.

2. REAGENTS


- Phosphate Buffered Saline (PBS) : Sodium Chloride, Potassium Chloride, Disodium Phosphate, Potassium dihydrogen phosphate
- Nucleic Acid Preservation : Nondisclosure

3. Characteristics

The color of STD/HPV media is clear and each product contain 2ml, 3ml of media in plastic tube (12ml tube).

4. Specimen Collection Procedures

- (1) After peel and open the pouch, collect specimen by using collection swab.
 - * Specimen collector has to hold the upper part of swab breakpoint, not lower part.
- (2) Collect appropriate amount of specimen coated on the surface of swab tip.
 - * The plastic applicator can be broken at breakpoint by excessive force.
- (3) After open the cap and put the swab specimen into the transport medium, carefully break the swab shaft against the breakpoint using both hands and discard the top portion of the swab shaft.
 - * Please be care careful when you break the breakpoint of swab without splash.
- (4) Recap the transport medium tube tightly and shake it homogenous.
- (5) Record the name of specimen, sex, age, number, date and etc.,
- (6) As soon as possible, transfer the tube containing specimen to the laboratory for analysis.
 - * If it is too late to deliver, store at 2~8°C and analyze in 72 hours.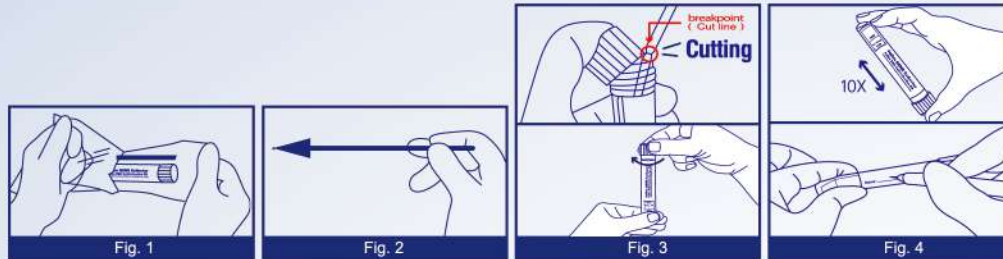

- [Fig.1] Peel open the sealed pouch pack.
 [Fig.2] Remove one swab from the pouch and collect the specimen without bending the swab.
 [Fig.3] Aseptically remove the cap from the vial and insert the swab into the tube containing medium.
 Break the swab shaft by bending the swab.
 [Fig.4] Replace the cap and secure the lid tight and record the patient's information on the label.
 Transfer the tube containing the specimen to laboratory for analysis.

5. Storage & Expiry Date


- Store : 1 - 25°C
- Expiry Date : 24 months after production

6. Package Unit

- 1pcs / individual package, 50pcs / Case

7. Precautions

- Do not freeze or heat before use.
- Please don't use after expiry date.
- Do not use if it is damaged, broken, contaminated or leaking.
- Please do not contact the reagent in tube.
- Single-use medical device.
- Do not reuse opened and used products.
- Record information on the label to deliver and analysis

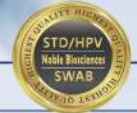STD/HPV Swab Collection for the Diagnosis of STD/HPV

ENDOCERVICAL SPECIMENS COLLECTION

- Peel open the STD/HPV swab pouch and hold the swab, the upper part of breakpoint. Please use a new swab if you touch the soft tip or lower part of breakpoint.
- By using the cleaning swab (Cat.No. NS-1), remove excess mucus from the cervical ostium and surrounding mucosa and discard this swab.
- Insert the specimen collection flocked swab (Cat.No. NFS-2 or NFS-7) into the endocervical canal and gently rotate the swab clockwise for 10 to 30 seconds to ensure adequate sampling.
 - * In the case of Endo/Exocervical swab (Cat.No. NFS-8), insert the swab L character into the ectocervix and gently rotate the swab clockwise for 10 to 20 seconds.
 - * In the case of cervical brush (Cat.No. CB-1), insert cervical brush into the cervical os, 1-1.5cm and rotate the cervical brush clockwise for 10 to 20 seconds.
 - * Please be careful, the swab can be breakable if you force too hard
- Withdraw the swab carefully and avoid any contact with the vaginal mucosa.
- After open the cap and put the swab specimen into the transport medium, carefully break the swab shaft against the breakpoint and discard the top portion of the swab shaft.
 - * When you don't break the breakpoint of the swab shaft (Cat.No. NFS-2, NFS-7, NFS-8, CB-1), put the swab specimen into the medium and shake the swab over 5 times to discharge samples of the swab tip into the medium.
 - And discard the swab.
- Recap the transport medium tube tightly and shake homogeneous.
- Record information about the patient on the label and as soon as possible, transfer tube containing specimen to the laboratory for analysis.

VAGINAL SWAB SPECIMENS COLLECTION

- Peel open the Vaginal Swab pouch and hold the swab, the upper part of breakpoint. Please use a new swab if you touch the soft tip or lower part of breakpoint.
- Carefully insert the swab into the vagina about 5cm past the introitus and gently rotate the swab 10 to 30 seconds.
 - Make sure the swab touches the wall the vagina so that moisture is absorbed by the swab.
 - Withdraw the swab carefully without touching the skin.
- After unscrew the cap from the tube, immediately place the swab specimen into the transport medium. Please take care not to spill the contents of the tube.
- Carefully break the breakpoint of the swab shaft with the breakpoint line at the top of the tube. And immediately discard the top portion of the swab shaft.
- Recap the transport medium tube tightly and record information about the patient on the label.
- As soon as possible, transfer the tube containing specimen to the laboratory for analysis.

RECTAL SWAB SPECIMENS COLLECTION

- Peel open the Rectal swab pouch and hold the swab, the upper part of breakpoint. Please use a new swab if you touch the soft tip or lower part of breakpoint.
- To collect the sample of mucosal surface, carefully insert rectal swab 3~5 cm into the anal canal and rotate the swab, at least three times, for 5 to 10 seconds against the rectal wall. And withdraw the swab carefully.
 - * If the swab is grossly contaminated with feces, discard the swab and use a new one.
- After remove the cap from the transport medium tube, immediately place the specimen swab into the transport medium. And carefully break the swab shaft against the top side of the tube at the breakpoint and be careful to avoid splashing of contents.
- Recap the specimen transport medium tube tightly and homogenize the sample medium by shaking from side to side.
- Record all of the information about the specimen on the label. And send it to the laboratory according to their specifications as soon as possible.

STD/HPV Transport Medium

for Sexually Transmitted Disease(STD)/Human Papilloma Virus(HPV)

Cervix-Swab™
Cervical Cyto-Brush, Endo/Exo Cervical Flocked Swab & Curette

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- Phosphate Buffered Saline (PBS) : Sodium Chloride, Potassium Chloride, Disodium Phosphate, Potassium dihydrogen phosphate
- Nucleic Acid Preservation : 비공개

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