


GST-4

Instructions for Use

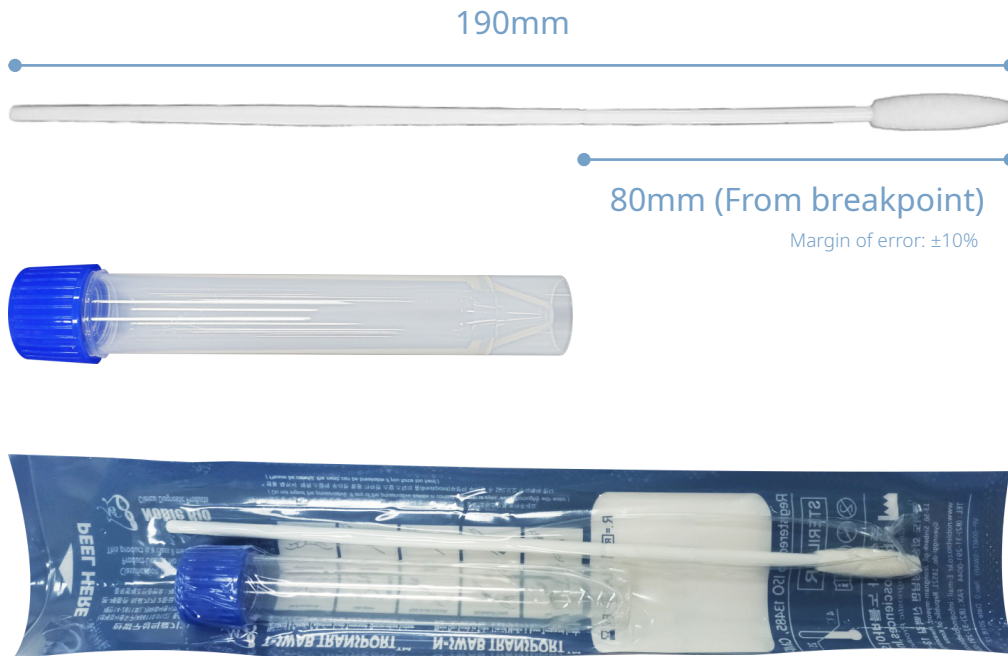

Vagina Specimen Collection method (Vaginal Sampling)

- (1) After opening the packaging, collect the sample using the NB Self-Drying Swab, which is a sample collection tool, as shown below. If you carelessly hold the lower part of the breakpoint, discard it and use a new one.
- (2) Take out the Collection Swab, carefully insert it about 5 cm from the mouth of the vagina, and gently rotate it 3-5 times (10-20 seconds) to collect the sample.
 - * Make sure that the swab comes into contact with the walls of the vagina and is absorbed by the swab head. Be careful not to touch other skin when removing the swab.
 - * In case of breaking the head of the swab during the subsequent test, be careful not to splash the contents of the sample. When dissolving in a reagent, the cap is tightly closed for accurate analysis and then vortexed to make the sample homogeneous.
- (3) On the tube label, write the sample name, sex, age, serial number, collection date, and type of sample.
- (4) The tubes in which the specimens are stored are kept at room temperature until transported to the laboratory.