N-SWAB TRANSPORT™ NB SELF-DRYING SWAB

How to use NB Self-Drying Swab

- [Picture 1] Open the pouch package and take out the tool.
- [Picture 2] Open the lid where the Swab is connected by holding the dotted line and turning it.
* It will not open if you hold the lower part and turn it.*
- [Picture 3] Take out the swab and collect specimens.
- [Picture 4] Put the sampled swab into the tube and close the lid for storage.

Warnings and Cautions

- This product is for in vitro diagnostic use only.
- All human specimens, blood derivatives, reagents, and materials used for sampling are considered capable of transmitting infectious diseases and are handled by professionals in a manner that prevents infection by laboratory personnel. Known testing methods do not provide a guarantee that there is no source of infection.
- Follow approved biohazard precautions and aseptic techniques. It should only be used by properly trained and qualified personnel. When handling any item contaminated with blood and other bodily fluids, standard precautions and institutional guidelines must be followed.
- Disinfect all biohazardous waste including specimens, containers and media after use.
- Class 1S means that a Swab can be used to sample body surfaces, bodily cavities (ex: oral, anus, vaginal and invasive surgical wounds).
- Do not re-disinfect unused swabs.
- Do not repackage.
- Do not use for any purpose other than the intended use.
- To use this product with a quick diagnostic kit or diagnostic device, the user must check its validity in advance.
- Do not use if the swab is visibly damaged (ex: the tip of the swab is broken).
- Do not use buffers or media to pre-wet the swab applicator.
- Tests must be carried out in an area with adequate ventilation.
- Safely dispose of containers and unused contents in accordance with national and local regulatory requirements.
- Wear suitable protective clothing, gloves, and eye/face protection when handling the contents of this kit.
- Wash your hands thoroughly after handling.
- Refer to the appropriate requirements/specifications for additional information on hazard symbols, safety, handling and disposal of components included in this kit.

CREATED DATE : 2021.05.04

경기도 화성시 정남면 신백길 13-50
13-50, SINBAEK-GIL, JEONGNAM-MYEON, HWASEONG-SI, GYEONGGI-DO, REPUBLIC OF KOREA
E-mail : info@noblebio.co.kr TEL : (82) - 31 - 291 - 0044
FAX : (82) - 31 - 291 - 0046 WWW.NOBLEBIO.CO.KR

NOBLE BIOSCIENCES, INC. IS THE COMPANY OF COLLECTION, TRANSPORT AND PRESERVATION SYSTEM FOR THE BIOPSY.

N-SWAB TRANSPORT™ NB SELF-DRYING SWAB


User Manual for NB Self-Drying Swab

Product name NB Self-Drying Swab
Cat. No SDST-100
Certificate No. 21-451

Intended Use

NB Self-Drying Swab [N-SWAB TRANSPORT™] is a tool for biopsy for collecting feces, saliva, DNA evidence from crime scenes, and STD bacteria and virus samples in the anus and vagina, and preserving and transporting them in a dry state.

Shape, Structure & Dimensions

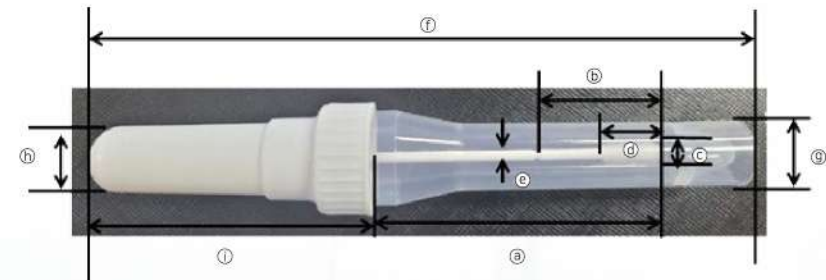- @ 90 mm (Length of Swab) | ⓑ 20 mm (Length of Breakpoint) | ⓒ 5.0 mm (Thickness of Swab head)
- @ 15 mm (Length of Swab head) | ⓔ 2.5 mm (Thickness of Swab stick) | ⓕ 161 mm (Full length)
- @ 15.5 mm (Thickness of Tube) | ⓓ 16 mm (Thickness of Cap) | ⓖ 70 mm (Length of Cap)

NB Flock Swab Applicator Type

No.	Swab Model	Product Description	Sampling Site
1.	NFS-2	Mini Tip(Head) Flocked Nylon Swab with Break Point (Length of swab, 90mm)	Fecal, Forensic, Saliva, Rectal, Vaginal and Buccal

Packing Unit

1 pc / Individual package | 30 pcs / Case | 300 pcs / Carton Box

Performance characteristics

NB Self-Drying Swab is for storing and transporting samples in a dried state by absorbing moisture by installing Silica Gel inside the tube. NB Self-Drying Swab is packaged in an aluminum pouch to prevent absorption of external moisture, and the finished product is sterilized by Gamma-Irradiation to prevent contamination of clinical samples.

The total length from the cap of NB Self-Drying Swab to the connected tube is 161 mm, the length of only the swab is 90 mm, and the length from the end of the swab head to the breakpoint is 20 mm. One flocked swab attached to the cap is placed inside the tube. To prevent contamination, make sure that the swab does not come into contact with foreign objects or the hands of the collector after collecting the sample, and put it in the tube as soon as possible.

Storage method and expiration date

This product is ready to use and requires no further preparation. Product should be stored in its original container at 1° C to 30° C until use. Do not overheat. Do not freeze before use. If it is stored improperly, its efficacy will decrease. The validity period (Exp. Date) is 3 years.

Specimen collection method

1. Feces Specimen Collection method (Feces Sampling)

1. After opening the package, collect the sample using the sample collection tool, NB Self-Drying Swab. When collecting the sample, the collector never grabs the lower part of the breakpoint marked on the swab while collecting the sample.

2. Put the entire tip of the swab into the feces and rotate it left and right 2~3 times to collect the sample. Collect the amount enough to be coated on the tip of the swab. Be careful as the breakpoint may break if excessive force is applied to the swab.

* The amount of specimen for culture (fecal) must comply with the amount of specimen collection required by the testing center.

* For molecular diagnosis (DNA/RNA) samples, about 0.2-0.3 g of samples are collected, and too few or too many samples may affect DNA/RNA quality and quantity during DNA/RNA extraction. => Determine the amount of sample according to the instructions for use of the DNA/RNA Extraction kit.

* In case of breaking the head of the swab during the subsequent test, be careful not to splash the contents of the sample. When dissolving in a reagent, the cap is tightly closed for accurate analysis and then vortexed to make the sample homogeneous.

3. Write the sample name, sex, age, serial number, collection date, and type of sample on the tube label.

4. Store the sampled tubes at room temperature until transported to the laboratory.

2. Rectal Specimen Collection Method (Rectal Sampling)

1. After opening the packaging, collect a sample as follows using a sample collection tool, NB Self-Drying Swab.

* When collecting a sample, the collector should never hold the lower part of the breakpoint marked on the swab while collecting the sample.

2. To collect a sample from the rectum, insert 2-3 cm of the head of the swab into the rectum and rotate it left and right 3 to 5 times to collect the sample. Be careful as applying excessive force to the swab can damage the swab breakpoint. The amount of sample collected for molecular diagnosis is about 0.2-0.3 g.

3. Carefully put the sampled swab into the transport tube so that it does not come into contact with the surrounding area, and close the lid.

* Be careful not to splatter the contents of the specimen when breaking the breakpoint of the swab during the subsequent test. For accurate analysis, make the sample homogeneous by vortexing after tightly closing the cap.

4. On the tube label, write the sample name, sex, age, serial number, collection date, and type of sample.

5. Store the sampled tubes at room temperature until transported to the laboratory.

3. Specimen Collection Method for Criminal investigation (Forensic Sampling)

Proper collection of samples from crime scenes is critical to clinical investigation results. Specimen Refer to the procedures described below or internal sampling procedures for specific guidance on collection procedures.

* Wear protective clothing, gloves (powder free) and goggles during sample collection and handling.

* When collecting a sample, hold the swab with a grip wearing sterile gloves.

* Sampling should be collected as soon as possible immediately after opening the packaging.

* Do not forcefully press, excessively press, or bend the swab when collecting samples in the field.

The swab's shaft has a molded breakpoint near the tip, which can deliberately break the swab, so be careful.

Sampling Method

1. Put on sterilized gloves, open the aluminum pouch, and take out the NB Self-Drying Swab.

2. After selecting the target for sample collection, open the white cap with Silica Gel installed inside.

3. Scrape the sample containing DNA 4-5 times in one direction with the tip of the Swab and collect it. At this time, the swab tip during sampling should only touch the sample collection part (target) to reduce the risk of cross contamination.

4. Put the sampled swab into the tube as quickly as possible and lock it tightly.

5. Write the sample name, sex, age, serial number, collection date, and type of sample on the tube label.

6. Keep the sampled tube at room temperature until it is transported to the laboratory.

4. Vagina Specimen Collection method (Vaginal Sampling)

(1) After opening the packaging, collect the sample using the NB Self-Drying Swab, which is a sample collection tool, as shown below. If you carelessly hold the lower part of the breakpoint, discard it and use a new one.

(2) Take out the Collection Swab, carefully insert it about 5 cm from the mouth of the vagina, and gently rotate it 3-5 times (10-20 seconds) to collect the sample.

* Make sure that the swab comes into contact with the walls of the vagina and is absorbed by the swab head. Be careful not to touch other skin when removing the swab.

* In case of breaking the head of the swab during the subsequent test, be careful not to splash the contents of the sample. When dissolving in a reagent, the cap is tightly closed for accurate analysis and then vortexed to make the sample homogeneous.

(3) On the tube label, write the sample name, sex, age, serial number, collection date, and type of sample.

(4) The tubes in which the specimens are stored are kept at room temperature until transported to the laboratory.

5. Buccal DNA/RNA Specimen collection method (Buccal Sampling)

The epithelial cells in the cheeks are collected and DNA/RNA tests are performed.

1. After opening the package, collect the sample using the NB Self-Drying Swab, which is a sample collection tool. If you carelessly hold the lower part of the breakpoint, discard it and use a new one.

2. Take out the swab and place it on the cheek side of the mouth, scratch it up and down, and sample for at least 20 seconds. For standard DNA collection, scratch the cheek firmly for about 1 minute to sample.

3. After collecting the sample, put it safely in the storage tube and lock it tightly. At this time, be careful not to let the swab head come in contact with foreign substances such as fingers.

4. On the tube label, write the sample name, sex, age, serial number, collection date, and type of sample.

5. Keep the sampled tube at room temperature until it is transported to the laboratory.

N-SWAB TRANSPORT™ NB SELF-DRYING SWAB

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